



*Examine Real Quality & Value™*

# OPERATION, CARE & MAINTENANCE

## *POWER MAT PLATFORMS*

*Models 252 & 253*




## THANK YOU

Thank you, for selecting and purchasing quality Clinton equipment. We are proud to serve your medical equipment needs with products that deliver value and offer superior performance. Please read these instructions carefully to insure your Clinton products are used and operated according to specifications and that they are maintained to Clinton's recommendations.

## SPECIFICATIONS

All models have a load capacity of 600 lbs.

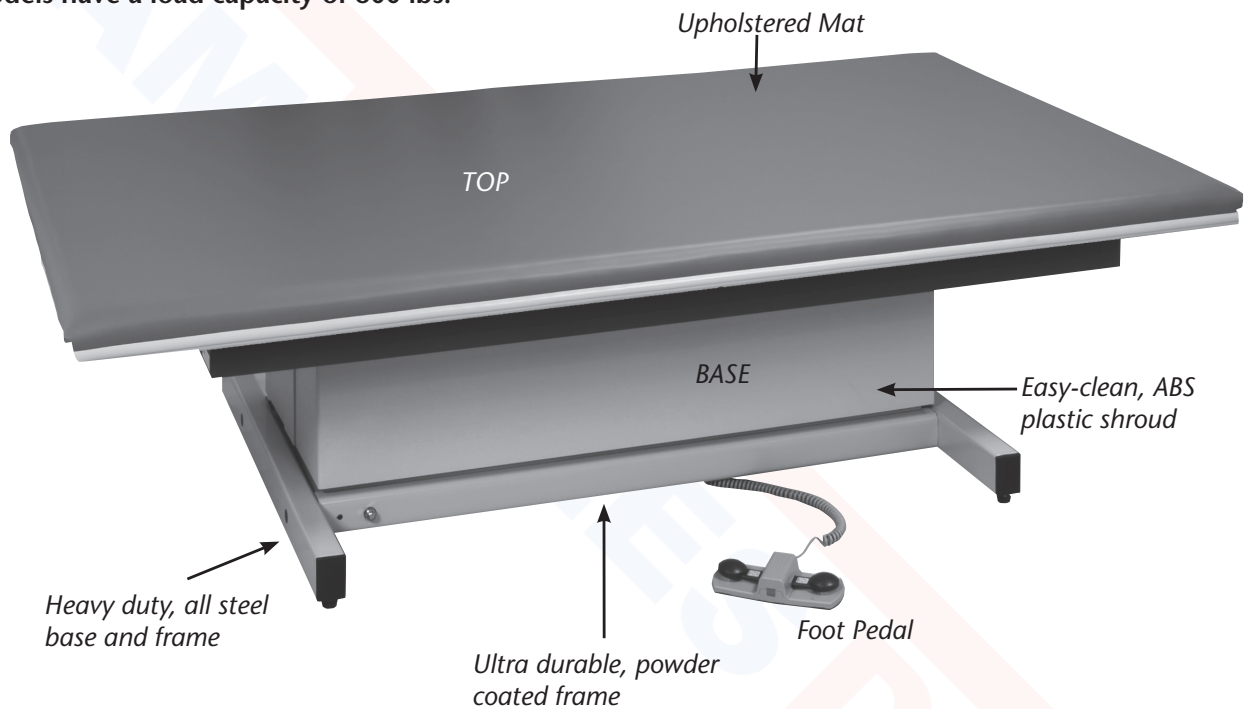

### Model Dimensions

#### Power Mat Platform

	Length	Width	Height Range
<b>252-47</b>	7'	4'	22"–34"
<b>252-57</b>	7'	5'	22"–34"
<b>252-68</b>	8'	6'	22"–34"

#### Power Mat Platform with Removable Mat

	Length	Width	Height Range
<b>252-47</b>	7'	4'	22"–34"
<b>252-57</b>	7'	5'	22"–34"
<b>252-68</b>	8'	6'	22"–34"

## Motor Specifications

---

LA34 is a technological state-of-the-art actuator that, due to its innovative construction can push up to 10,000 N at a speed of 5 mm/sec. and with a power consumption of approx. 7 Amp.

The actuator is approved according to EN 60601-1 and UL 2601-1

### LA34 Motor Power Ratings

			Push	Speed
120VAC	7 amps	60 hz	10,000N	5 mm/sec

## CLEANING GUIDELINES

Clinton uses only premium vinyl for upholstery, however, care should be taken when cleaning to ensure that the upholstery or color is not affected by the cleaning agent. Use only mild cleaning agents on upholstery. Clinton recommends using a 1 to 1 mix of ammonia and water or a 1 to 4 mix of bleach and water, rinse thoroughly with clean water and pat surface dry. This is not a guarantee and does not relieve the user from responsibility of the proper and safe use of all cleaning agents. Test on inconspicuous area prior to using any cleaning agent on upholstery's visible surface. Certain cleaning agents can be harmful to the surface.

## WARRANTY

Clinton Industries warrants manufactured products to be free of defects in materials, workmanship or design under normal use for three years on all mechanical products and two years on all electric equipment, from the date of purchase. This warranty does not include damage resulting from accident, abuse or misuse, alteration, or negligence of the product and expressly excludes normal wear of parts (including but not limited to vinyl). Contact Clinton for full warranty details.

## ADJUSTMENTS

### REMOVABLE MAT *(model 252 only)*

---



The mat on model 252 is held in position with Velcro® strips at the corners. To remove or replace the mat, simply lift the corners and detach the Velcro®.

For assistance with issues not covered in this guide, or questions about your Clinton Power Mat Platform, please call Clinton Industries at 800-441-9131.



*Examine Real Quality & Value™*

1140 East Edison St., York, PA 17403  
800-441-9131 • 717-848-3519  
Toll-Free Fax 866-522-1233  
Fax 717-843-5871  
[www.clinton-ind.com](http://www.clinton-ind.com)