230

Universal Procedures Chair

Ritter®

[English](#)

[Español](#)

[Français](#)

User's Guide

Shop tables & accessories at

TP202 20-42-FO-00014 Rev A1 C2169 003-1500-99 Rev HA6 (1/30/23) Style P

Product Information

(The information below is required when calling for service)

Dealer [name / phone]:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company [name / phone]:

**Model / Serial Number
Location**

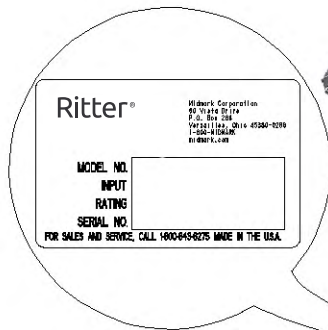
Table of Contents

Important Information

| | |
|--|---|
| Safety Symbols | 4 |
| Intended Use..... | 5 |
| Electrical Requirements | 5 |
| Electromagnetic Interference | 5 |
| Disposal of Equipment | 5 |
| Transportation / Storage Conditions..... | 5 |
| Authorized Representatives | 6 |
| Safety Instructions..... | 7 |
| Hand / Foot Control Connection..... | 7 |

Operation

| | |
|--|----|
| Power To The Chair | 8 |
| Base Up / Down | 9 |
| Back Up / Down..... | 10 |
| Foot Up / Down | 11 |
| Tilt Up / Down..... | 12 |
| Paper Roll..... | 13 |
| Stirrups | 14 |
| Treatment Pan..... | 15 |
| Duplex Receptacle (optional - not available on 230 volt models)..... | 16 |
| Headrest Positioning | 17 |
| Knee / Chest Position..... | 18 |
| Rotational Base (optional)..... | 19 |

Maintenance

| | |
|---------------------------|----|
| Cleaning | 20 |
| Periodic Maintenance..... | 20 |
| Calling for Service | 20 |

Troubleshooting

| | |
|---------------------------|----|
| No Foot Up Function | 21 |
|---------------------------|----|

Specifications

| | |
|--|----|
| Weights, Dimensions, Electrical Specifications | 22 |
| Model Identification / Compliance Chart..... | 23 |
| Range of Motion & Dimensions..... | 24 |

Warranty Information

| | |
|------------------------|----|
| Limited Warranty | 26 |
|------------------------|----|

Important Information - Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which could result in injury if not avoided.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice, or condition.

Refer to manual

Proper Shipping Orientation

Fragile

Type B, Applied Part

Protective earth ground

Keep dry

Catalogue Number

Serial Number

Manufacturer

Maximum stacking height
(palletted units)

Fuse rating specification

Protected against dripping water

Duty Cycle (motor run time):
30 seconds ON, 5 minutes OFF

Pan must be fully retracted to
enable Foot Up function.

Patient weight limit

This product has been evaluated with respect to electrical shock, fire & mechanical hazards only, in accordance with UL60601-1 and CAN/CSA C22.2 NO. 601.1.

Intended Use

This product is intended to be used as a chair / table to provide positioning and support of patients during general examinations and procedures conducted by qualified medical professionals.

Electrical Requirements

Caution

When using high frequency surgical devices or endocardial catheters:

- *Use non-conductive material to insulate patient from metal portions of the chair.*
- *Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.*

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This Midmark chair is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this chair:

- Remove interfering device from room
- Plug chair into an isolated circuit
- Increase separation between chair and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Transportation / Storage Conditions

Ambient Temperature Range..... +41°F to 100°F (+5°C to +38°C)
Relative Humidity..... 10% to 90% (non-condensing)
Atmospheric Pressure 500hPa to 1060hPa (0.49atm to 1.05atm)

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading

Midmac Roundabout

Opposite Al Mana Petrol Station

Doha, Qatar

Tel: +974-44337000

Fax: +974-44337100

Safety Instructions

Caution

Be sure that all personnel and equipment are clear of the chair before activating any function. Failure to do so could result in personal injury.

Hand / Foot Control Connection

There are two hand / foot control cord inlets. One is located on the base column at the foot end of the chair, and one is located on the PC board cover at the head end. These cord inlets are interchangeable.

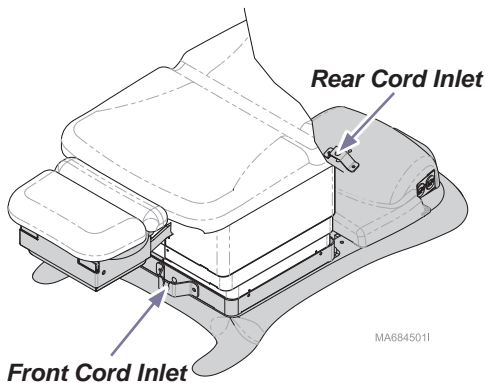
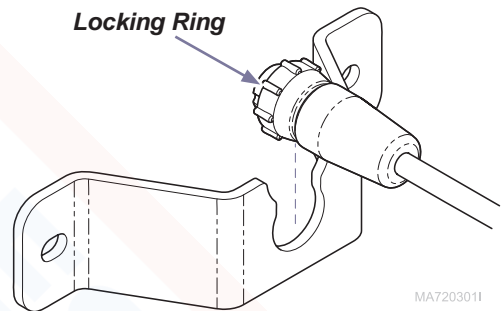
Equipment Alert

Never plug hand / foot control into any other outlets. Doing so may result in damage to the equipment.

To connect hand / foot control cord...

- A) Slide narrow portion of cord connector into bracket.
- B) Connect cord to inlet (see Note).
- C) Tighten locking ring.

Note: The cord connector is "keyed" and must be oriented properly to connect.

Operation

Power To The Chair

If the chair has no power...

All functions will be inoperable.

Before calling for service...

- A) Check power cord connections.
- B) Check / reset facility circuit breaker.
- C) Check / replace chair fuse(s).

If Power Indicator Light is...

ON: Chair has power.

OFF: No power to chair.

Before replacing fuses...

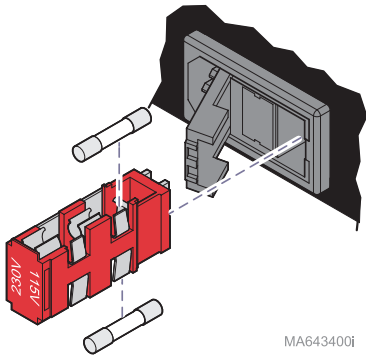
Disconnect power cord from chair.

To replace fuses...


[Refer to Specifications section for correct fuse size & ratings]

- A) Open the fuse holder cover.
- B) Remove fuse holder & faulty fuse(s).
- C) Install new fuse(s), then replace fuse holder.

Attention: Voltage setting that shows thru window of fuse holder cover must match facility supply voltage.

MA643400i

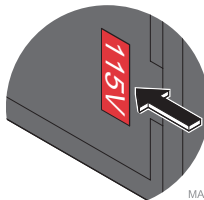MA644600i

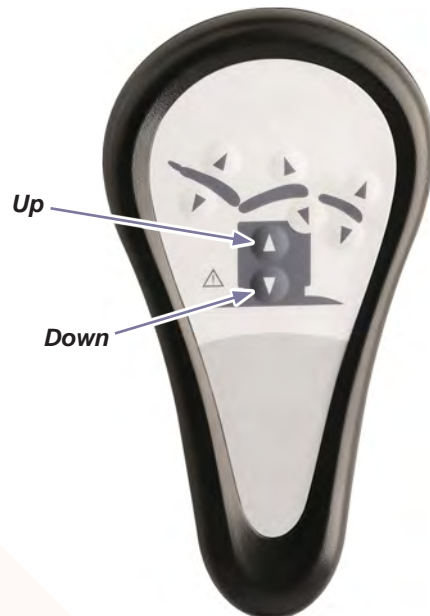
Equipment Alert

Incorrect voltage setting will result in damage to the chair.

Base Up / Down

To move the Base UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Up / Down button is released.
Maximum / minimum position is reached.

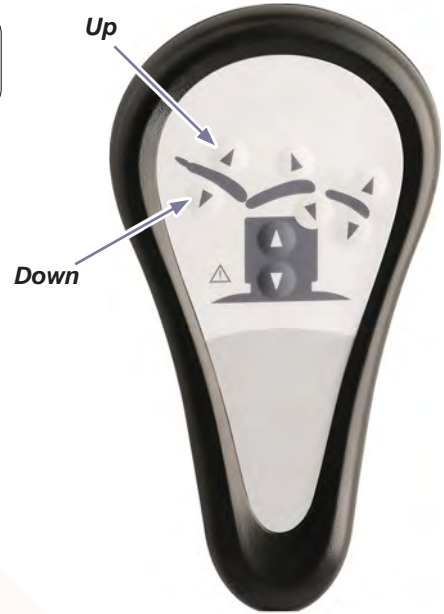
Max.
Standard base: 40 in. (101.6 cm)
Rotational base: 43.5 in. (110.5 cm)

Min.
Standard base: 22.5 in. (57 cm)
Rotational base: 26 in. (66 cm)

Back Up / Down

To move the Back section UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.

Foot Up / Down

Attention!

The Foot Up function is disabled when the treatment pan is pulled out.
(You will hear several “beeps” when Foot Up is activated with the pan pulled out.)

To resume Foot Up operation:
Push the pan all the way in.
Press Foot Up button.

To move the Foot section UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.

Max.
 $0^{\circ} \pm 0.5^{\circ}$

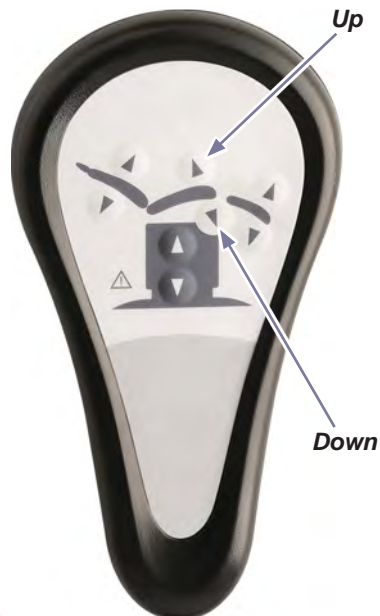
Min.
 $90^{\circ} \pm 1^{\circ}$

Tilt Up / Down

To Tilt the seat section UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.

Max.
 $30^{\circ} \pm 2^{\circ}$

Min.
 $0^{\circ} \pm 0.5^{\circ}$

Paper Roll

Paper Roll Size (max.)..... 18 in. long x 3.5 in. diameter
(45.7 cm x 8.9 cm)

To install the paper roll...

- A) Pivot paper roll cover back to open.
- B) Install paper roll, then pull paper over chair.
- C) Pivot paper roll cover forward to close.

MA642601I

Stirrups

Caution

Never use stirrups to lift chair.

Be sure the stirrups are locked in place before using.

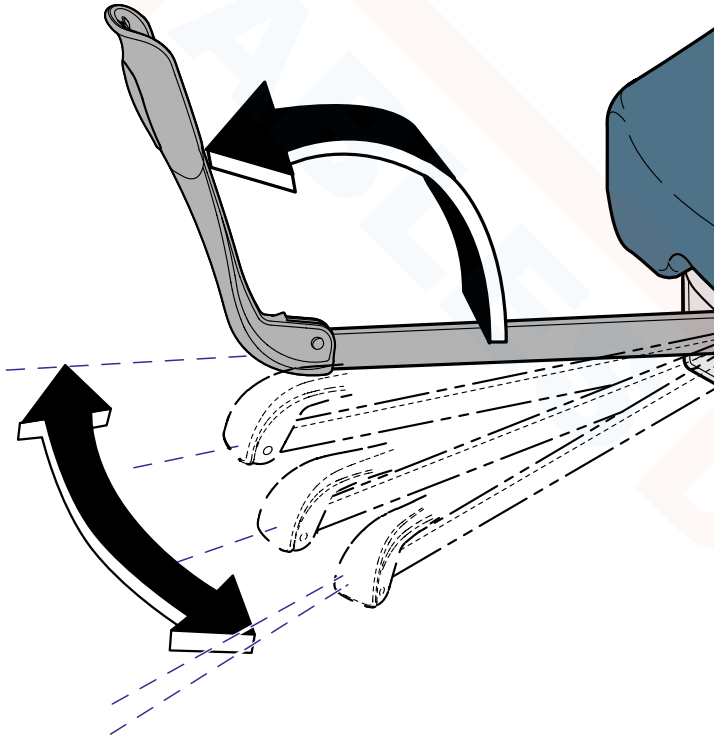
The stirrups will not support the patient's entire weight / excessive force.

To position the stirrups...

A) Pull the stirrup out, then unfold.

B) Lift the stirrup slightly, then move it left or right as desired.

C) Release stirrup to lock in position.

MA602500i

Treatment Pan

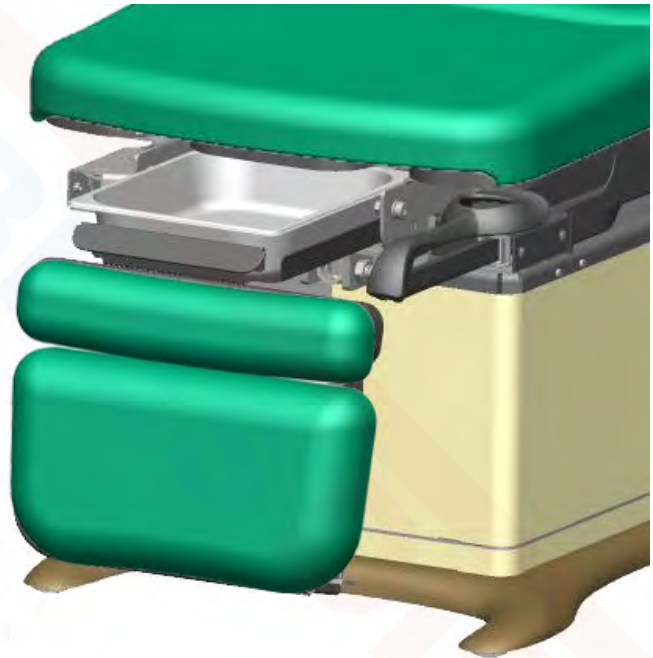
Attention!

The Foot Up function is disabled when the treatment pan is pulled out.
(You will hear several “beeps” if Foot Up is activated with the pan pulled out.)

To resume Foot Up operation:
Push the pan all the way in.
Press Foot Up button.

To access the treatment pan...

- A) Lower the foot section.
- B) Pull the treatment pan slide out.

Duplex Receptacle (optional - not available on 230 volt models)

The duplex receptacle provides power for accessories used during medical procedures. Power to the receptacle will be interrupted if the maximum load is exceeded.

Maximum Load [both receptacles combined] 115 VAC, 3 amps, 50/60 Hz

Equipment Alert

Exceeding the maximum load for the receptacle may result in blown fuses. If this occurs, all chair functions will be disabled until fuses are replaced. [Fuse replacement procedure is shown on the **Power To The Chair** page.]

Headrest Positioning

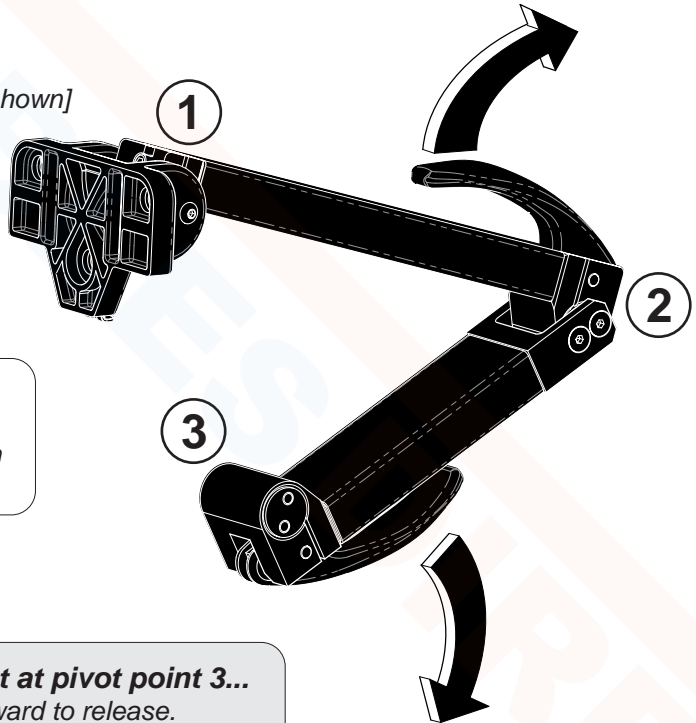
Caution

Be sure headrest is locked securely in position before use.

To adjust the headrest at pivot points 1 and/or 2...

- A) Pull upper handle outward to release.
- B) Move pivot point(s) to desired position.
- C) Push upper handle inward to lock in place.

[For clarity, the upholstery is not shown]

Note

The handle tension is adjustable.
Contact your service provider if an
adjustment is required.

To adjust the headrest at pivot point 3...

- A) Pull lower handle outward to release.
- B) Move pivot point to desired position.
- C) Push lower handle inward to lock in place.

MA642200i

Knee / Chest Position

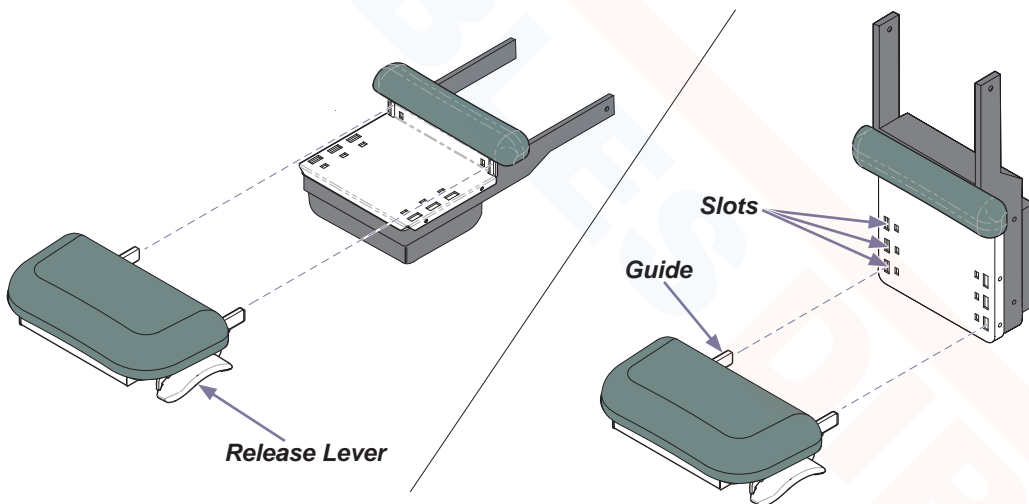
Caution

Be sure foot section is locked securely in position before use.

To remove the foot section...

- A) Lift release levers on either side of foot section.
- B) Pull foot section straight out.

MA6847001

To install foot section in Knee / Chest Position...

- A) Lower the foot section.
- B) Insert foot section guides into desired slots.
- C) Be sure foot section is securely engaged before using.

Rotational Base (optional)

To rotate the chair base...

- A) Press and release the brake pedal.
- B) Rotate the chair to desired position.

To lock in position...

- A) Press and release the brake pedal.

Note: The base will automatically lock in position after two minutes.

Maintenance

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted / Plastic Surface

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

Calling for Service

[Product serial number is required]

Contact your Midmark dealer, or log onto www.midmark.com to locate your nearest service provider.


[To contact](#) Midmark directly:

1.800.Midmark (1.800.643.6275) or 937.526.3662
8:00 am until 5:00 pm. Monday through Friday (EST) [excluding standard US holidays]

Troubleshooting

No Foot Up Function

| <i>Function</i> | <i>Status</i> | <i>Problem</i> | <i>Correction</i> |
|---------------------|---|--------------------------|------------------------|
| Foot Up |  | No movement ("beeps") | Push treatment pan in. |
| All other functions |  | OK | - |

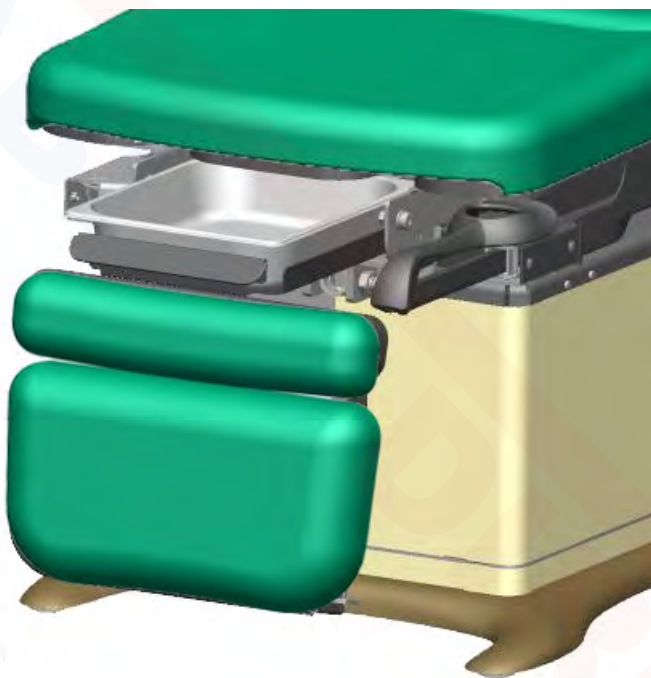Specifications

Weights, Dimensions, Electrical Specifications

| | |
|---|--|
| Patient Weight (<i>maximum</i>): | 450 lbs (204.1 kg) |
| Paper Roll (<i>maximum size</i>): | 18 in. long x 3.5 in. diameter (45.7 x 8.9 cm) |
| Range of Motion & Dimensions: | [See Range of Motion & Dimensions page] |
| Weight of Chair: 230 (w/standard base) w/packaging & skid 230 (w/rotational base) w/packaging & skid | 455 lbs (206.4 kg) 510 lbs (231.3 kg) 540 lbs (244.9 kg) 600 lbs (271.1 kg) |
| Power Cord: | 8 ft. (243.8 cm) long |
| Electrical Requirements: | [See Compliance Charts] |
| Foot Control Voltage: | 10 VAC, SELV (Safety Extra Low Voltage) |
| Duplex Receptacle (optional) maximum load: | 115 VAC, 3A, 50/60 Hz |
| Fuses: <i>Located at power cord inlet:</i> F1 & F2 | 6.3 A, 250V, Type T, 5 x 20 mm |
| Duty Cycle (motor run time): | Intermittent Operation [30 seconds ON / 5 minutes OFF] |
| Classifications: | Class 1, Type B Applied Part |
| Protection against ingress of fluids: <i>Foot Control <u>only</u>:</i> | Ordinary Equipment IPX1 |
| Regulatory Compliance: | [See Compliance Chart] |

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Model Identification / Compliance Chart

| Model | Description | Serial Number Prefix | Complies To: | | | | Electrical Ratings: | | |
|---------|--|----------------------|--------------|--------------------------|-------------|--------------------|----------------------------|------------|-------------|
| | | | UL 60601-1 | CAN/CSA 22.2, #601.1-M90 | IEC 60601-1 | EN 60601-1-2 (EMC) | VAC +/-10% | Amps | Cycles (Hz) |
| 230-001 | Ritter Universal Proc. Chair (Non-Programmable) | V | X | X | X | X | Configurable 115 230 | 7 3.5 | 50 / 60 |
| 230-002 | Ritter Universal Proc. Chair (Non-Programmable) w/ Receptacle | V | X | X | X | X | 115 | 10 | 50 / 60 |
| 230-003 | Ritter Universal Proc. Chair (Non-Programmable) w/ Rotational Base | V | X | X | X | X | Configurable 115 230 | 7.5 3.8 | 50 / 60 |

Range of Motion & Dimensions

Max.

Standard base: 40 in. (101.6 cm)
Rotational base: 43.5 in. (110.5 cm)

Min.

Standard base: 22.5 in. (57 cm)
Rotational base: 26 in. (66 cm)

Max.
30° ±2°

Min.
0° ±0.5°

Max.
85° ±3°

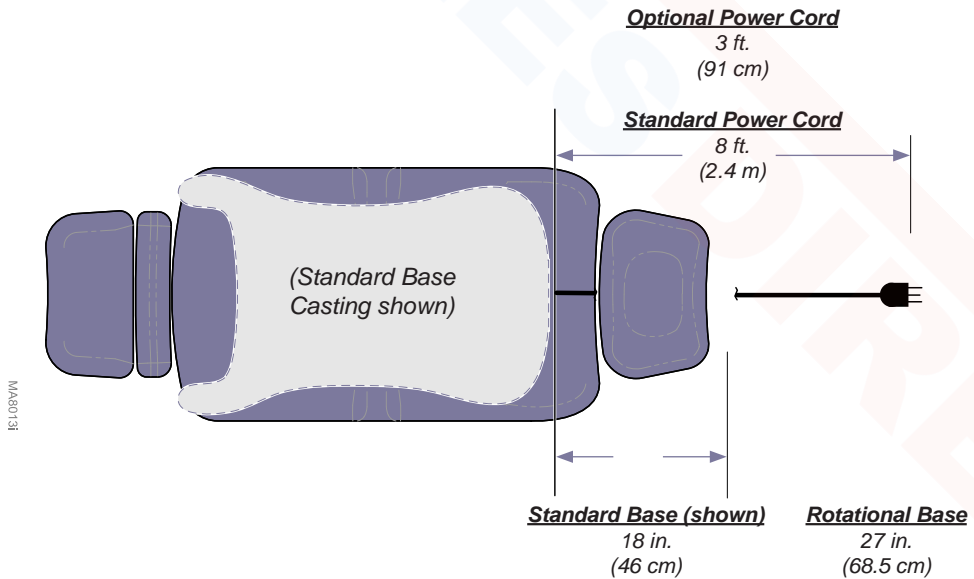
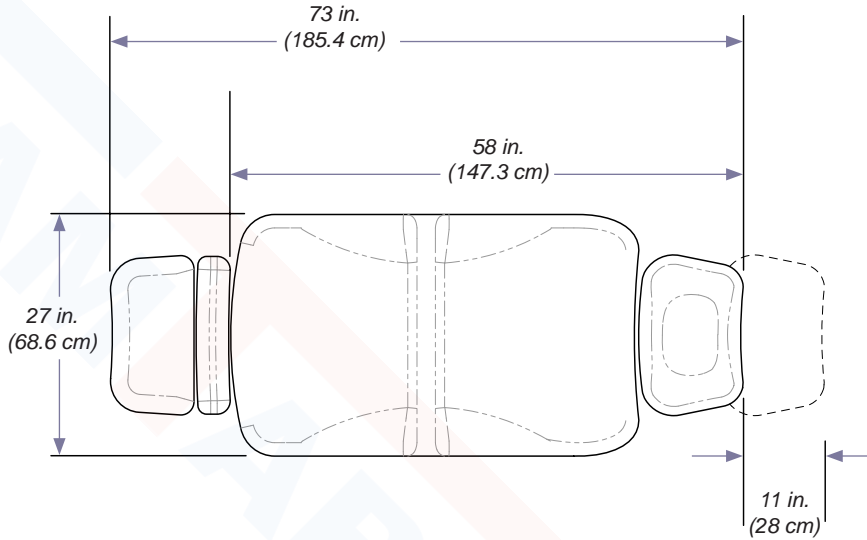
Min.
0° ±0.5°

Max.
0° ±0.5°

Min.
90° ±1°

Range of Motion & Dimensions - (continued)

Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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