# Ritter®

## 95 Treatment Table

[English](#)  
[Español](#)  
[Français](#)

**Applies to Models:**

95-001

**Special Tools:**

none

## Set Up / User Guide

Style P

003-1649-99 Rev. AA2 (11/10/21)

TP202 20-42-FO-00014 Rev A2 C2169

Shop tables & accessories at


# Product Information

**Dealer:**

**Date of Purchase:**

**Model / Serial Number:**

**Midmark Authorized Service Company:**

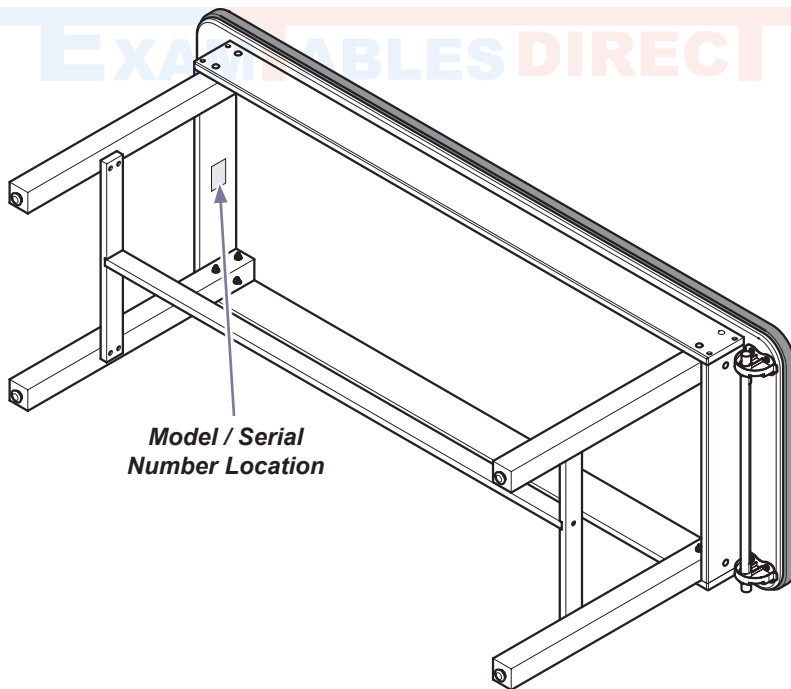**Model / Serial  
Number Location**

# Table of Contents

## **Important Information**

<i>Product Information</i> .....	2
<i>Intended Use</i> .....	4
<i>Safety Instructions</i> .....	4
<i>Product Registration</i> .....	4
<i>Transportation / Storage Conditions</i> .....	4
<i>Disposal of Equipment</i> .....	4
<i>Important Information</i> .....	5
<i>Authorized Representatives</i> .....	6

## **Set Up**

<i>Set Up Instructions</i> .....	8
----------------------------------	---

## **Operation**

<i>Operating Instructions</i> .....	9
-------------------------------------	---

## **Maintenance**

<i>Cleaning</i> .....	10
<i>Preventative Maintenance</i> .....	10
<i>Calling for Service</i> .....	10

## **Warranty Information**

<i>Limited Warranty</i> .....	11
-------------------------------	----

## ***Intended Use***

This product is intended to be used as a table to provide positioning and support of patients during examination procedures conducted by qualified medical professionals in an office environment.

## ***Safety Instructions***

### **WARNING**

***No modification of this equipment is allowed.***

## ***Product Registration***

To register your product, go to [www.midmark.com](http://www.midmark.com)

## ***Transportation / Storage Conditions***

Ambient Temperature Range:.....-30°C to +60°C (-22°F to +140°F)  
Relative Humidity..... 10% to 90% (non-condensing)  
Atmospheric Pressure .....500hPa to 1060hPa (0.49atm to 1.05atm)

## ***Disposal of Equipment***

At the end of product life, the table, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

# Important Information - Safety Symbols

## **WARNING**

*Indicates a potentially hazardous situation which could result in serious injury.*

## **Caution**

*Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.*

## **Equipment Alert**

*Indicates a potentially hazardous situation which could result in equipment damage.*

## **Note**

*Amplifies a procedure, practice, or condition.*

## Symbol Glossary

Fragile

Keep dry

Patient Weight Limit

Maximum stacking height  
(palletted units)

Manufacturer

Catalogue Number

Serial Number

# Authorized Representatives

## **WARNING**

***In the event of any serious incident in relation to the device,  
please contact Midmark and the appropriate competent authority.***

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India

Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)  
Mumbai, 400022  
Tel:+91 22 4915 3000  
Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd  
Abdullah M. Al Khalifi Street  
Al Nakheel Dist. II  
P.O. Box 116105  
Jeddah (East)  
21391  
Saudi Arabia  
Tel: + 966 2 286 4707  
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.  
Al Barsha,  
Dubai, UAE  
Phone: +971 4 501 4000  
Fax: +971 4 501 4100

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading  
Midmac Roundabout  
Opposite Al Mana Petrol Station  
Doha, Qatar  
Tel: +974-44337000  
Fax: +974-44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.  
HaTnufa St 7  
Petah Tikva, Israel  
Tel: +972 3-925-4040  
Fax: +972 3-924-9977

## ***Authorized Representatives - continued***

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,


Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

# Set Up

## Assembly Instructions

- A) Lay table top assembly upholstered side down on a clean surface.
- B) Secure leg assemblies with carriage bolts & nuts.  
Note: Do not tighten nuts at this time.
- C) Secure cross brace with drywall screws.
- D) Tighten nuts securing leg assemblies.
- E) Install four leveling screws.
- F) Install paper roll brackets & dowel rod as shown.  
Note: There are pre-drilled holes at one end. Install roll brackets as shown to allow notch access for paper roll refills.

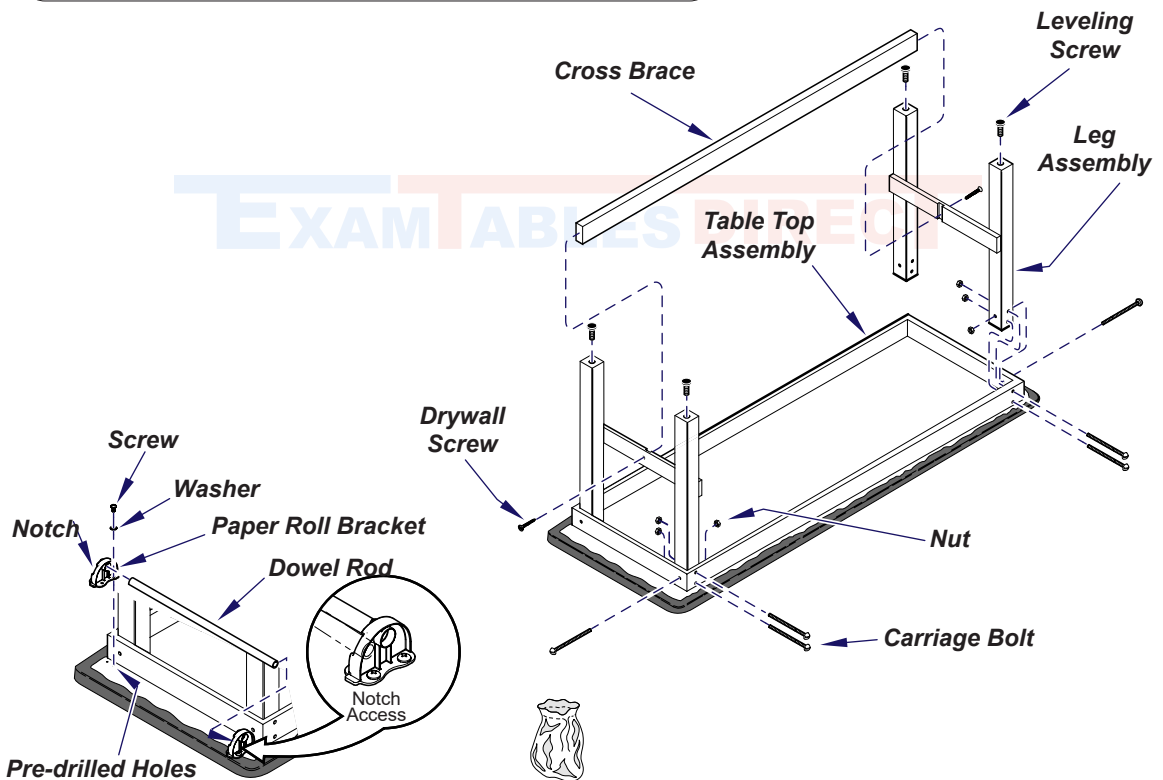Parts Bag contains the following parts:

- |                        |                         |
|------------------------|-------------------------|
| Leveling Feet (4pcs)   | Carriage Bolts (12 pcs) |
| Drywall Screws (2 pcs) | Nuts (12 pcs)           |
| Screws (8 pcs)         | Washers (8 pcs)         |

# Operation


## **WARNING**

To avoid injury, do not put head or extremities under the paper tear strap.

### **To install paper tear strap**

Slide strap over table as shown.

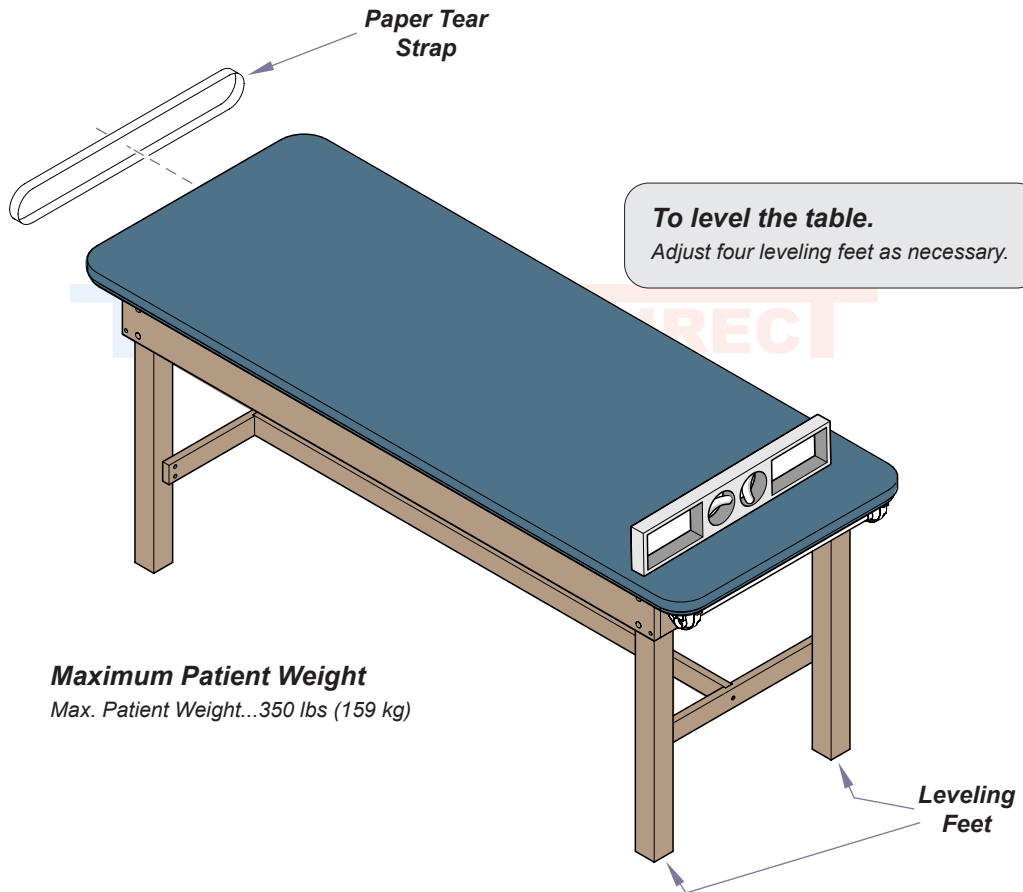# Maintenance

## Cleaning

### Upholstery

#### Equipment Alert

*The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.*

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [www.midmark.com](http://www.midmark.com) in the Technical library under the User Information tab for your product.

### Wood / Plastic Surfaces

Clean the wood and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

## Preventative Maintenance

Periodically inspect the following areas:

- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

## Calling For Service

If service is required, contact your authorized Midmark dealer.

### Note

*Model / Serial Number information is required when calling for service.*

To contact Midmark directly:

1.800.MIDMARK (1.800.643.6275)  
8:00 am until 5:00 pm Monday - Friday (ET) [excluding standard US holidays]

## **Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty**

**SCOPE OF WARRANTY** Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

**APPLICABLE WARRANTY PERIOD** The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

**OBTAINING WARRANTY SERVICE** Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

**EXCLUSIONS:** This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

**SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.**

**EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER** MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

**STATUTE OF LIMITATIONS** No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

**NO AUTHORIZATION** No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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© Midmark Corporation 2021