

USER MANUAL

OAKWORKS® Treatment Tables



Powerline Series
Backrest



Seychelle Series
Flat Top

www.oakworks.com · 717.235.6807



Notice

The information contained within this document is subject to change without notice and should not be construed as a commitment by OAKWORKS®, Inc.

OAKWORKS®, Inc. encourages requests for technical specifications and the like documentation to ensure accuracy. The appropriate documentation is available upon request.

OAKWORKS®, Inc. shall not be liable for incidental or consequential damages in connection with or arising out of the furnishing, performance, or use of this document and the program material which it describes.

Printed in U.S.A.

All rights are reserved. No part of this document may be photocopied, reproduced or translated to another language without prior written consent of OAKWORKS®, Inc.

OAKWORKS® is a registered trademark of OAKWORKS®, Inc.

TABLE OF CONTENTS

- Product Use Description 1
- Symbol Identification 1
- Important Safety Instructions 1
- Product Description & Photo
 - Powerline Series 2-3
 - Seychelle Series 4-5
- Installation
 - Table Assembly 6
 - Inset Shelf 6
- Directions for Use
 - Paper Roll Holder Use 7
 - Wave Backrest Use 7
 - Table Extender 7
 - QuickLock Face Rest 8
- Cleaning & Disinfection
 - Recommended Cleaners/Disinfectants 9
 - Cleaning Process 9
- Inspections 10
- Warranty Information 10
- Model Number & Serial Number 11
- Specifications
 - Product Specifications 12
 - Environmental Conditions 12
- Contact Information back cover

PRODUCT USE DESCRIPTION / IMPORTANT SAFETY INSTRUCTIONS

PRODUCT USE DESCRIPTION

The Oakworks Powerline & Seychelle Tables are used to support the patient during examinations and procedures. They are intended to be operated by a professional in an exam/treatment environment. No special training is required but a review of the following manual is important for the safety of the operator and patient. The professional should read and understand this entire manual before use with a patient. There are no known contraindications to the use of this equipment.

SYMBOL IDENTIFICATION



This symbol, when used in this manual and on product labels, represents a caution warning. Be sure to read and comply with all precautions and warnings.



This symbol, when used in this manual and on product labels, indicates the name and address of the manufacturer.



This symbol, when used in this manual or on product labels, indicates the country of manufacture along with date of manufacture of the device next to it.

IMPORTANT SAFETY INSTRUCTIONS



CAUTION

READ AND SAVE THESE INSTRUCTIONS

- The Backrest section is not designed to support the entire weight of the patient. Do not sit on the Backrest section.
- The table is designed to be a stand alone table. This table must not be modified or incorporated into any other equipment.
- There are potential pinch points around and underneath the tabletop. It is the responsibility of the operator of this equipment to insure that bystanders are not in the area below or around this equipment during operation.
- Proper operation of this equipment is very important for the safety of the operator, patient, and any other individuals in the area of this equipment. Directions for use of this equipment are described in this manual. The operator should read these sections carefully.
- Do not use outdoors.
- Use this furnishing only for its intended use as described in this manual. Do not use attachments not recommended by Oakworks.
- **Weight Limit: (patient and accessories) 500 lbs. (227 kg.) Do not exceed.**

PRODUCT DESCRIPTION

Powerline Flat Top



STANDARD SPECIFICATIONS	
Height	Fixed 30" (76 cm.)
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	24" (61 cm.), 27" (69 cm.), or 30" (76 cm.)
Length	72" (183 cm.)
Padding	2.5" (6 cm.) Semi-Firm
Paper Roll Holder	Accommodates up to 21" (53 cm.) rolls (roll not included)
Table Weight*	90 lbs. (41 kg.)
OPTIONS & ACCESSORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 23 lbs. (10.5 kg.))	
Padding - 1.75" (4 cm.) Firm Response	

PRODUCT DESCRIPTION

Powerline Wave Backrest



STANDARD SPECIFICATIONS	
Height	Fixed 30" (76 cm.)
Wave Backrest	Manual 0-60°, 6 set positions
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	24" (61 cm.), 27" (69 cm.), or 30" (76 cm.)
Length	72" (183 cm.)
Padding	2.5" (6 cm.) Semi-Firm
Paper Roll Holder	Accommodates up to 21" (53 cm.) rolls (roll not included)
Table Weight*	90 lbs. (41 kg.)
OPTIONS & ACCESSORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 23 lbs. (10.5 kg.))	
Padding - 1.75" (4 cm.) Firm Response	

PRODUCT DESCRIPTION

Seychelle Flat Top



STANDARD SPECIFICATIONS	
Height	Fixed 30" (76 cm.)
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	24" (61 cm.), 27" (69 cm.), or 30" (76 cm.)
Length	72" (183 cm.)
Padding	3" (8 cm.) Comfort
Table Weight*	90 lbs. (41 kg.)
OPTIONS & ACCESSORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 23 lbs. (10.5 kg.))	
Padding - 4" (10 cm.) Comfort	

PRODUCT DESCRIPTION

Seychelle Wave Backrest



STANDARD SPECIFICATIONS	
Height	Fixed 30" (76 cm.)
Wave Backrest	Manual 0-60°, 6 set positions
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	24" (61 cm.), 27" (69 cm.), or 30" (76 cm.)
Length	72" (183 cm.)
Padding	3" (8 cm.) Comfort
Table Weight*	90 lbs. (41 kg.)
OPTIONS & ACCESSORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 23 lbs. (10.5 kg.))	
Padding - 4" (10 cm.) Comfort	

INSTALLATION

TABLE ASSEMBLY

TOOLS REQUIRED:

9/16" / 14mm Wrench/Socket

SUPPLIED PARTS:

TOOLS


- 1 - Large Allen Wrench
- 1 - Small Allen Wrench

LEG HARDWARE

- 8 - Reinforcement Plates
- 16 - Spring Washers
- 4 - Long Large Bolts
- 4 - Lock Nuts



LEG BRACE

- 1 - Wood Leg Brace
- 2 - Short Bolts
- 2 - Barrel Nuts

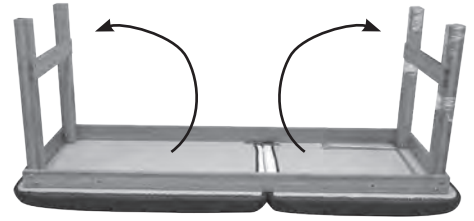

The tables are shipped complete in one box. Follow the step below to fully assemble the table:

1. Remove the table from the box and place it down on the floor with the top down. Remove all hardware and the Leg Brace.
2. Unfold the legs to a vertical position and remove any protective wrappings.
3. Install the long large bolts, reinforcement plates, spring washers and lock nuts as shown, two bolts on each leg. Using the large Allen wrench and 9/16" / 14mm wrench socket, tighten all 8 bolt/nut connections to 12 ft-lbs minimum or until the spring washers are flat.
4. Install the barrel nut into the leg brace being sure the screw slit is horizontal. Install the short bolt through the leg rail. Start the bolt into the barrel nut using your fingers first. After the bolt has started, tighten the bolt using the small Allen wrench. Make sure the leg brace is flush with the adjoining rails while tightening bolts. Repeat on opposite end of the leg brace.
5. Turn your table right side up. Get up on the table and rock back and forth to check that all bolts are tight. The table should be very stiff. Dependent on the environmental conditions of your table, it may be necessary to periodically re-tighten the hardware.

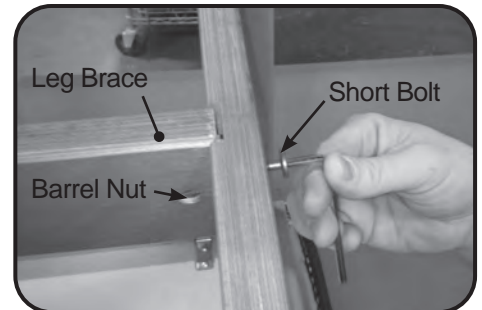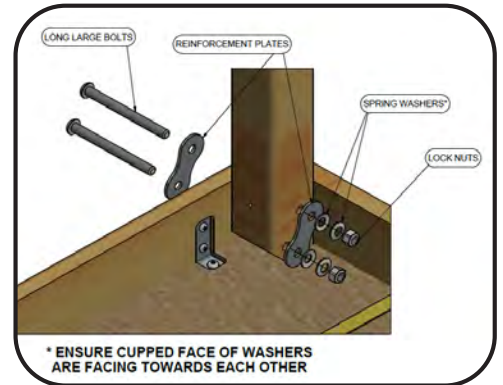

SHELF ATTACHMENT



CAUTION


- The Maximum Weight Capacity for the Shelf is 100 lbs. (45 kg.)

If you have ordered a shelf, drop the shelf into place. The shelf has grooves at both ends that enable it to be held firmly in place by the legs. Further stabilize the shelf by installing the enclosed Velcro™ strips. Place strips on each corner of the shelf's bottom side, and on each end of the rails. Make sure not to mismatch Velcro™ strips; place all "hook" or all "loop" pieces on either the rail or shelf. The shelf is reversible and can be used on either side.




DIRECTIONS FOR USE

PAPER ROLL HOLDER USE

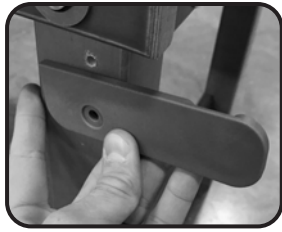

After the table is set-up, locate the hole in the leg at the head end.

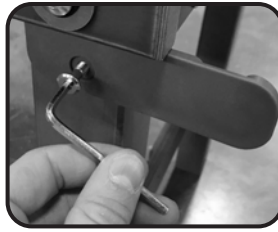

Using the supplied screw & Allen wrench, secure the paper roll holder bracket into place. Repeat step for the opposite side.



Locate the wooden dowel & feed it through your paper roll. Place the wooden dowel in the groove located in each of the paper roll holder brackets.



To secure the wooden dowel, feed it into the pocket of the groove on both sides of the table.

WAVE BACKREST USE



CAUTION

Do not adjust the Backrest with a patient on the table.



TO RAISE THE BACKREST

Lift the backrest to the desired height and release. The Backrest support arm will lock in the nearest slot in the side rail.



TO LOWER THE BACKREST

Lift the backrest slightly, lift and hold the support arm out of the slot and lower the backrest to the desired position.


TABLE EXTENDER



CAUTION

Do not put excessive weight or pressure on the table extender.

The Table Extender lengthens the table for your tallest patients and can improve access to your patient.

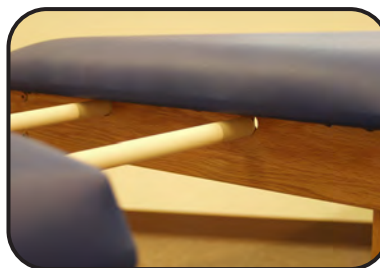

Locate the Accessory attachment outlets.



Unfold the dowels and insert into the outlets.

DIRECTIONS FOR USE


QUICKLOCK FACE REST



CAUTION

- Do not put excessive weight or pressure on the Quicklock Face Rest.
- Do not extend the dowels of the Quicklock Face Rest 3" (7.6 cm.) from the table top.
- The Maximum Weight Capacity for the QuickLock Face Rest is 25 lbs. (11 kg.)

Double articulating action provides nearly limitless positioning options with the QuickLock Face Rest. The adjustable pad takes pressure off sensitive facial areas while providing superb support. The QuickLock Face Rest can be used in both prone and supine positioning and folds out of the way when not in use. Adjustments are made easily by loosening the cam locks, adjusting for height and angle and then locking the cam locks into the desired position.

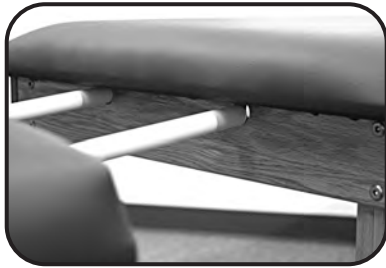

Important Safety Note:
Maximum Distance from edge of
Table Top= 3" (7.6 cm.)



Insert dowels into
Accessory attachment
outlets.



Loosen Cam Locks &
adjust face rest into place.



Tighten the Cam Locks.



The Face Rest can be
used for either prone
or supine positioning.

CLEANING & DISINFECTION

RECOMMENDED CLEANERS/DISINFECTANTS

Reference the Recommended Cleaners and Disinfectant list (MMINML0008-EN) that came with the table. This information can also be found at www.oakworksmed.com under product information.

All cleaners and disinfectants have the ability to degrade the upholstery to some extent. However, following the recommended cleaner and disinfectant list and cleaning process will provide the best care for your table and support a long product life.

Oakworks recommends a prepackaged wipe for cleaners/disinfectants to ensure best distribution of disinfectant for the required kill time, without leaving excess residue and/or overexposing components therefore minimizing the potential for damage to materials. Please read and follow disinfectants manufacturers' directions for cleaning and disinfection.

Oakworks does NOT recommend the use of cleaners/disinfectants containing Hydrogen Peroxide, Acetic Acid, or Phenolics. These chemicals can cause damage to the appearance and/or material integrity of various components. Also, while the recommended cleaners/disinfectants list includes products containing Quaternary Ammonium compounds ("quats"), not all products containing quats are approved for use. Some contain additional detergents and/or surfactants which can damage some materials.

Use of non-approved cleaners or disinfectants may lead to damage to upholstery and other materials found on the table and will void the warranty.

CLEANING PROCESS

Follow the cleaners/disinfectant manufacturers' directions for use. Please note that cleaning and disinfecting an Oakworks table is a two part process. First it must be cleaned of any visible soil, then it can be disinfected. Oakworks recommends that the table be positioned in the flat position during the cleaning process. Please follow this procedure for best results:

1. Using an approved cleaner or mild liquid soap and water, clean any visible soil off of the table, working from the top to the bottom of the table. It is recommended that the upholstery be cleaned at least once a week to prevent disinfectant build up.
2. Rinse with clean water and dry with a clean cloth or towel.
3. Using an approved disinfectant, thoroughly disinfect all surfaces of the tabletop and any high-contact areas such as handles, handsets, etc., making sure they remain wet for the disinfectant manufacturer's recommended contact time. Do not allow disinfectant to pool on the upholstery after the recommended contact time.
4. Wipe off any excess liquid with a cloth or towel and clean water.
5. Dry all surfaces with a clean cloth or towel.

Avoid using writing instruments or other similar instruments around the upholstery as it can cause permanent staining. If this does occur, do not wipe with an alcohol based cleaner. Instead, blot the stain with a clean cloth/ paper towel. Use a recommended cleaner or disinfectant to remove the stain. Follow this with a rinse of clean water.

INSPECTIONS / WARRANTY

INSPECTIONS

RECOMMENDED REGULAR INSPECTIONS (monthly or local standard)

- Visually inspect components for obvious damage that could cause problems during operation.

RECOMMENDED PERIODIC INSPECTIONS (yearly or local standard)

- Visually inspect components for obvious damage that could cause problems during operation.
- Check that all fasteners are present and fastened securely.
- Clean unusual buildup of dirt on the table and/or parts of the table not normally cleaned on a regular basis.
- Check for tears or cracks in the upholstery.

WARRANTY


View complete warranty details at www.oakworksmed.com under the information tab “Warranty and Return Policy”

MODEL NUMBER & SERIAL NUMBER

MODEL & SERIAL NUMBERS



The model number and serial number are located under the top of the table.

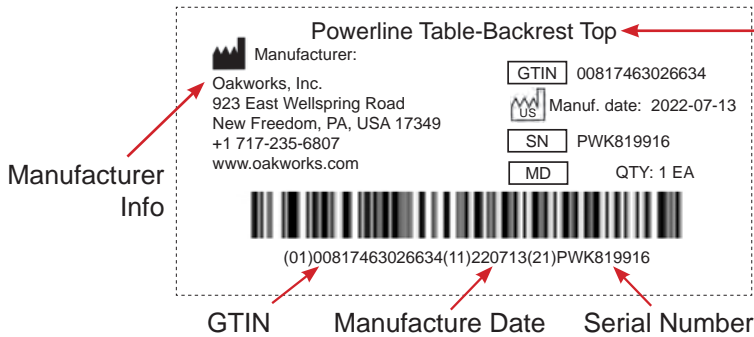

Model Name

Unique Device Information:

- GTIN - 14 digit number unique for each variation of a model
- Manufacture Date - Country of manufacture and date the device goes into production in YYYY-MM-DD format
- SN - Serial Number
- MD - Medical Device Symbol
- QTY - Quantity of the product

SPECIFICATIONS

PRODUCT SPECIFICATIONS

Weight	90-113 lbs. (41-51) kg.
Weight Limit	500 lbs. (226 kg.)

ENVIRONMENTAL CONDITIONS

Conditions	Temperature	Humidity	Atmospheric Pressure
Normal Use	50° (10°C) to 104° (40°C)	20% to 60% RH	98 to 105 kPa
Storage & Transport	-20° (-29°C) to 135° (57°C)	20% to 95% RH	98 to 105 kPa

USER MANUAL

OAKWORKS® Treatment Tables

CONTACT INFORMATION:



OAKWORKS® Inc.

923 East Wellspring Road
New Freedom, PA USA 17349

Phone: 717.235.6807

FAX: 717.235.6798

www.oakworks.com

Manual Part Number MMMNST0005-EN / Med-RA-PM-44

Revision level: 10

Revision Date: 12/16/2022

English, Printed in USA

